

amoena

RECOVERY CARE/AMOCURA
CURASCAR GARMENTS ABDOMEN / LEGS

Instructions for use.

Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, the name Amoena has been synonymous with innovative and high-quality products that support the particular needs of women following breast surgery.

Intended purpose

Scar compression garments are intended to be used to achieve good scar healing through the application of even, extensive and continuous pressure. The therapeutic use of scar compression garments causes a reduction in perfusion and an increase in tissue elasticity. The aim is to reduce the risk of impaired scar formation or achieve its regression. The goal of compression therapy is a flattening of the scar, an increase in tissue elasticity and a reduction in scar rigidity.

Indication

Damage to the skin and to the healing function of the skin, with impaired scar formation

- following reconstructive or aesthetic surgery
- to improve scar/wound healing to the point of complete scar healing
- to stabilise the results of surgery

Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV, decompensated heart failure, septic phlebitis, phlegmasia cerulea dolens, acute erysipelas, acute bacterial, viral or allergic inflammations, untreated or advanced skin diseases/contact eczematata, untreated open wounds, swellings or extremities of an unknown origin, untreated malignant tumours.

Relative contraindications

Pronounced weeping dermatoses, malignant lymphoedema, peripheral arterial occlusive disease (PAOD) stage I/II, peripheral neuropathy in diabetes mellitus, sensory disorders of the extremities, compensated heart failure, skin infections, skin diseases (e.g. blistering dermatoses), intolerance to pressure or ingredients of the product, primary chronic polyarthritis. If there are open wounds in the application area, you must first cover the wound area with an appropriate wound pad or a dressing before applying the scar compression garment.

Side effects

There are no known side effects with correct use. However, should negative changes such as skin irritation occur during prescribed use, please contact your doctor or medical retailer immediately. If there is known intolerance to one or more ingredients of this product, please consult your doctor before use. The manufacturer is not liable for damage/injuries that have arisen due to improper handling or incorrect use.

General care instructions

Please note the care instructions on the care label attached. If present, all fastenings must be closed. We recommend turning the scar compression garment inside out before washing. Wash the scar compression garment separately the first time, in order to prevent colour bleeding. You should wash your scar compression garment daily at 30°C using the delicate washing programme or fine wash cycle. A laundry net bag will provide additional protection for your scar compression garment. Please do not use fabric softener! If washing by hand, please rinse thoroughly and do not wring out. The drying time can be shortened by placing the scar compression garment on a thick towel after washing, rolling it and pressing it firmly. Next, hang the scar compression garment up loosely. Please do not leave it in the towel and do not dry on a radiator or in the sun. The scar compression garment must not be chemically cleaned or dried in the tumble dryer.

Scar compression garments

Art. No.	Article
45000	Panty 2-ZIP-BO (Nude)
45001	Panty 2-ZIP-BO (Black)
45002	Bermuda 2-ZIP-BO (Nude)
45003	Bermuda 2-ZIP-BO (Black)
45004	Belly bandage 3-VE (Nude)
45005	Belly bandage 3-VE (Black)

Instructions for use

Your medical retailer will be happy to show you the correct application. Wear your scar compression garment continuously for 23 hours or as directed by your doctor. Rough, unfiled fingernails and/or sharp-edged rings, etc., can damage the very fine material.

Only wear the scar compression garment as prescribed by a doctor. The scar compression garment should only be issued by trained specialist personnel in the medical retail trade. Continued treatment success or a change in the clinical picture warrant a new measurement and, if necessary, selection of the correct size of the garment. For this reason, a check by the medical retailer is recommended. If there is any damage, please contact your specialist retailer. Please do not repair it yourself, as this can impact on quality and medical effect. For hygiene reasons, unsuitable for the treatment of more than one patient.


Instructions for closing the Belly bandage 3-VE three-part Velcro fastening:

Size S at the far ridge – size M at the centre ridge

Size L at the far ridge – size XL at the centre ridge

Size 2XL at the far ridge – size 3XL at the centre ridge

All three-part Velcro fastenings should ideally align vertically.

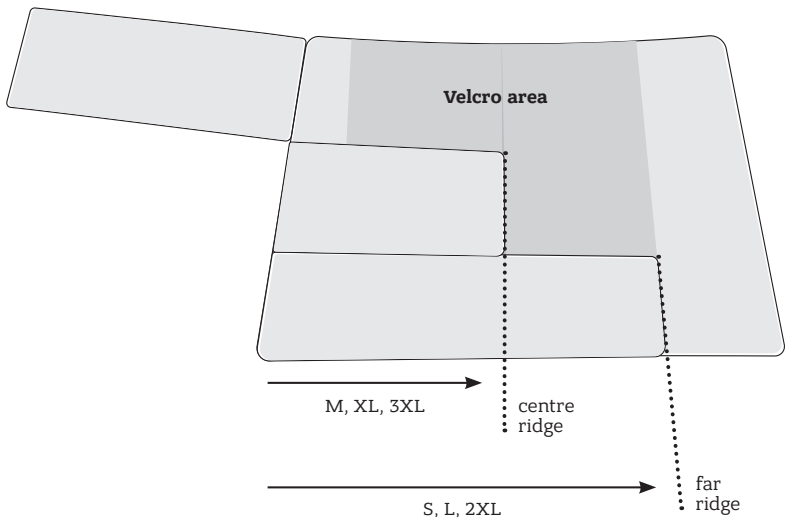

Material composition

Panty 2-ZIP-BO	polyamide, elastane, cotton
Bermuda 2-ZIP-BO	polyamide, elastane, cotton
Belly bandage 3-VE	polyamide, elastane, polyester, cotton

For material composition by percentage, see care label. Free from latex, silicone and nickel.

Storage instructions and shelf life

Store in a dry place and protect from sunlight. The useful life of the scar compression garment is six months and is based on correct handling, e.g., in terms of maintenance, application, removal and use of the products within the normal scope.

Special instructions



Protect from sunlight



Store in a dry place



Single patient – multiple use

Disposal instructions

You can dispose of your scar compression garments in the usual household waste. You do not need to observe any special criteria for disposal.

Reporting obligation

If you get any side effects or have any problems with this product, please contact Amoena at the contact details below.

Limited liability for defects

Product liability refers to the lack of defects / faultlessness of the purchased item at the time of sale to the enduser. The seller is liable for all defects existing at the time of sale, even if 'hidden' defects are showing only after a certain time. The liability for defects counts for 24 months after purchase.

Contact us on **1800 773 285** or **CustomerService.AmoenaAustralia@amoena.com** with any queries you may have

Service information

Do you have any questions?

If so, we would like to help – please call us or write to us at:
(03) 9230 5200 or CustomerService.AmoenaAustralia@amoena.com

www.amoena.com.au

amoena

Amoena is an ISO 9001 / 13485 / 14001 certified company.

ID 1287 - 04/21

Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.

© 2020 All rights reserved Amoena Australia. 107 Fergie Street Fitzroy North, Vic 3068



Amoena Australia
107 Fergie Street Fitzroy North, Vic 3068
www.amoena.com.au

