amoena

ACCESSORIES

LEISURE FORM | NIPPLE SET |
AQUA WAVE

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsanvisning. Instruksjoner for bruk. Användningsinstruktioner. Instrukcja użycia. Instrução de uso. Käyttöohjeet. Návod k použití. Инструкции по применению. Οδηγίες χρήσης. 使用説明. 使用説明書.
مادخت سالا تامي لعن

Deutsch	DE.....	3
Englisch	ENG.....	8
Français	FR.....	13
Español	ES.....	18
Italiano	IT.....	23
Nederlands	NL.....	28
Dansk	DK.....	33
Norsk	NO.....	38
Svenska	SE.....	43
Polski	PL.....	48
Portugal	PT.....	53
Suomalainen	FI.....	58
Český	CZ.....	63
Русский	RU.....	68
Ελληνικά	GR.....	73
中國	CN.....	78
日本語	JP.....	83
88	ARAB	العربية

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. For more than 45 years, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.

Indication

Amoena accessories create a visual balance after breast surgery.
Target group: Women after various breast surgeries.

Contraindications/Risks

No contraindications or risks are known or expected when using the following products. In rare cases allergic reactions may occur. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin.

LEISURE FORM

Art. No.	Article
132N	Leisure Form

Designated Purpose

Breast shaper made of lightweight foam for occasional use during leisure time or overnight.

It ensures visual body symmetry after a mastectomy and keeps the remaining breast in position when lying down/sleeping.

Material Composition

Weighted foam: polyurethane, barium sulphate (7727-43-7)
Cover: polyamide, elastane

General Instructions for Use

- This foam form is not a substitute for wearing a silicone breast form, which is necessary for weight compensation and an upright posture.
- It is not suitable for sports activities.
- The form can be worn in a pocketed top, bra or nightdress to restore body symmetry.
- Detectors at the airport may react to the barium sulphate filling material.

General Care Instructions

- The Amoena Leisure Form is washable.
- Remove the fabric cover from the form before washing.
- Only clean the foam form with a damp cloth. Do not launder in a washing machine or place in water.
- Wash the fabric cover either in the washing machine on a delicate wash cycle or by hand with a mild detergent at a maximum of 30°C.
- Lay the cover flat to dry – do not place in the tumble dryer.
- As soon as the fabric cover is completely dry, turn the cover inside out. Lay the front of the cover on the front side of the form and then carefully pull the back of the cover over the form.
- You can then put the breast shaper back in the pocket of your Amoena sleepwear or a well-fitting bra

AQUA WAVE

Art. No.	Article
149	Aqua Wave

Designated Purpose

Transparent silicone breast shaper for all activities in and on the water.

Material Composition

- Polyurethane film
- Polydimethylsiloxane and silicone oil

General Instructions for Use

- The Amoena Aqua Wave can be worn in the integrated bra pocket of a bathing costume or bikini.
- This breast shaper is not a longterm substitute for wearing a silicone breast form, which is necessary for weight compensation and an upright posture..

General Care Instructions

- For the care of your Amoena Aqua Wave, we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed in clean water and dried with a soft towel.
- Please handle your Aqua Wave with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can destroy them. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Discolouration does not impair either the quality or function of the breast shaper.
- Some substances can damage the film coating of the breast shaper:
 - Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
 - Ethyl acetate and other esters
 - Acetone

Wait until the substances have been absorbed and wash your hands before you touch your breast shaper. Disinfectants are permitted.

NIPPLES

Art. No.	Article
134	Nipple Set Demo Kit
136	Nipple Set Ivory
137	Nipple Set Almond
138	Nipple Set Bronze
139	Nipple Set Tawny

Designated Purpose

The self-adhesive Amoena nipples complete the natural silhouette and provide maximum security combined with exceptional comfort.

Material Composition

Polydimethylsiloxane

General Instructions for Use

- Determine the position for the nipple and place it on the breast form, breast shaper or directly onto your skin. The adhesive material used is long-lasting and skin-friendly.
- It can be removed and reapplied without any problems.
- Thanks to the soft, comfortable material, Amoena nipples adapt perfectly to the natural rounding of the breast or breast form. Any unevenness from putting on the Amoena nipples may be corrected by gentle stroking.

General Care Instructions

- Careful handling and regular care are crucial for the long-term preservation of the adhesive area of the nipples. Daily cleaning reactivates the adhesive strength and is therefore the most important prerequisite for the secure fit of the nipples.
- The Amoena Soft Cleanser contained in the set is designed for cleaning and enables optimum adhesion. Other cleaning products can reduce the adhesive strength.
- To clean, add a drop of Amoena Soft Cleanser to the previously moistened adhesive surface of the nipple. Then foam up by hand and rinse with plenty of lukewarm water. Now leave the nipple to air dry or use a hairdryer on a low temperature setting.
- Please store the nipple on the carrier film provided.

Special Instructions

Protect from sunlight

Store in a dry place

Single patient – multiple uses

Breast form

Disposal Instructions

Dispose of the products in household waste.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies.

Service Information

Do you have any questions?

If so, we would like to help – please call us or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2022 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd