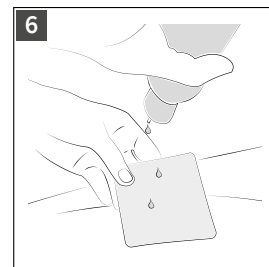
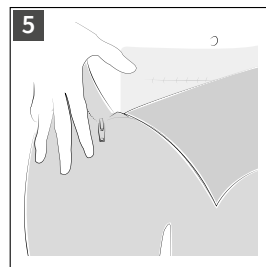
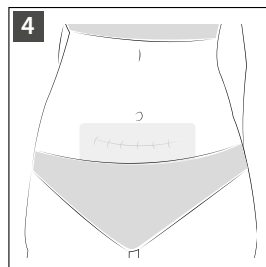
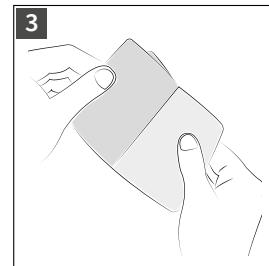
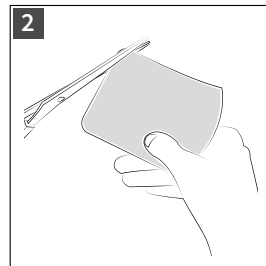
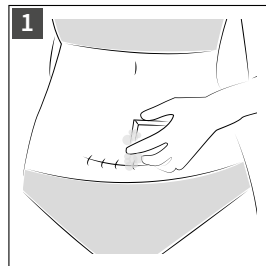
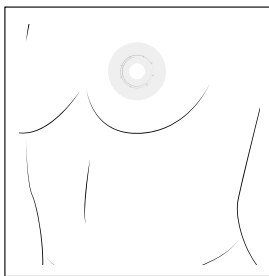
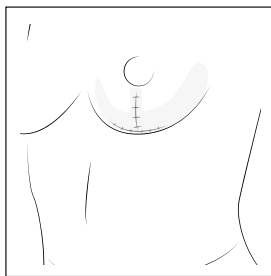
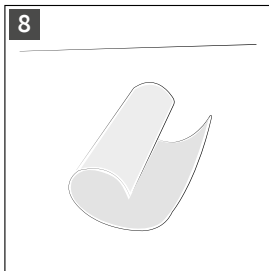
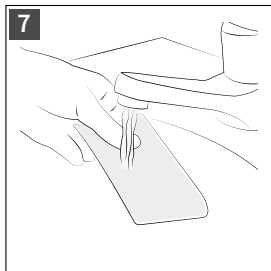
amoena

RECOVERY CARE CURASCAR SILICONES

Gebrauchsanweisung. Instructions for use. Mode d'emploi.
Instrucciones de uso. Instrução de uso. Istruzioni per l'uso.
Gebruiksaanwijzing. Brugsanvisning. Instruksjoner for bruk.
Användningsinstruktioner. Instrukcja użycia. Käyttöohjeet.
Návod k použití. Инструкции по применению.
Οδηγίες χρήσης. 使用説明. 使用説明書. مداخلت سال تامي لعنت

Deutsch	DE.....	5
Englisch	ENG.....	8
Français	FR.....	11
Español	ES.....	15
Italiano	IT.....	19
Nederlands	NL.....	23
Dansk	DK.....	27
Norsk	NO.....	31
Svenska	SE.....	33
Polski	PL.....	36
Português	PT.....	40
Suomi	FI.....	44
Česky	CZ.....	47
Русский	RU.....	50
Ελληνικά	GR.....	54
中國	CN.....	58
日本の	JP.....	62
66.....	ARAB	العربية

Anchors

Mammilla Circles

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.

Intended purpose

In scar therapy, silicone and compression provide the ideal combination. In compression treatment, a therapeutically effective pressure is also exerted on the closed wound area. In this way, excessive connective tissue remodelling can be counteracted.

The combination of constant surface pressure from the compression garment, along with silicone, has a positive influence on scar healing. The silicone creates a moist environment on the scar, which prevents the scar from drying out. It keeps the scar soft and has a positive effect on scar tissue.

Indication

- Prevention and treatment of impaired scars
- Treatment of red and raised scars
- After surgical procedures (e.g. after reconstructive or aesthetic surgery)
- Treatment recommended in combination with compression garments

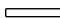
Contraindications

- Do not use on open wounds
- Do not use with skin irritation
- Do not use with known intolerance to the ingredient.
- Should skin intolerance occur during use, please do not use the silicone scar patch anymore.

Side effects

No side effects are known in the event of proper use. In rare cases, residual risks such as allergic reactions may occur. However, should negative changes, such as skin irritation occur during prescribed use, please contact your doctor or medical retailer immediately. If there is known intolerance to one or more ingredients of this product, please consult with your doctor before use. If your symptoms worsen during application, please consult your doctor immediately. The manufacturer is not liable for damages/injuries that have arisen due to improper handling or non-intended use.

SILICONE SCAR PLASTER

	Art. No.	Article	Packaging Unit	Dimensions
	010 (1)	Strips	1 pcs	10.24 x 1.38 inch
	010 (4)	Strips	4 pcs	10.24 x 1.38 inch
	011	Squares	2 pcs	10.24 x 3.94 inch
	012	Mammilla Circle	2 pcs	Ø outside 2.95 inch, Ø inside 0.79 inch
	013	Anchor	2 pcs	9.06 x 5.71 inch

All silicone scar patches bear the colour designation 'clear'.

Material composition

Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene cover strips

Instructions for use

- Your medical specialist will be happy to show you how to use it correctly. Amoena silicones should be worn according to the doctor's prescription.
- First clean the scar and surrounding skin thoroughly. Keep the scar tissue dry and free of grease. (Fig. 1)
- If necessary, you may cut the silicone scar patch into the desired shape and size. Make sure to cut shapes that are as round as possible, without corners. (Fig. 2).
- Remove the protective film from the silicone scar patch (Fig. 3).
- Carefully apply the self-adhesive side of the silicone scar patch onto the scar (Fig. 4).
- For an optimum result, the scar must be covered completely with the silicone scar patch. Ideally, this overhangs the scar by at least 1 – 2 cm.
- Now you can carefully pull your compression garment over the silicone scar patch (Fig. 5).
- When removing, please ensure that the silicone scar patch is pulled off from different areas.
- Increase the duration of application slowly. The silicone pad can be worn for up to 23 hours a day, after becoming familiar with it. To achieve an optimal treatment result, the pad should be worn for at least 12 hours every day. Amoena silicones are intended to be used 4-6 weeks in terms of their medical efficacy. The efficacy depends on the proper usage of the product within its usual extent.
- CuraScar silicones are suitable from 18 years on.
- Possible folds, caused by the packaging, do not affect the quality of the product.
- Avoid direct contact with perfume, hot surfaces and hot water.

Care instructions

- Please clean the silicone Pad daily.
- Clean the silicone scar patch with a few drops of the Amoena Soft Cleanser or a non-greasy soap (e.g. mild dishwashing liquid) and wash thoroughly by hand once every day (Fig. 6).
- Rinse the silicone scar patch under warm running water (Fig. 7).
- The silicone scar patch can roll up while still wet (Fig 8). The adhesive strength disappears completely.
- To dry, place the rolled-up silicone scar patch on a smooth surface, with the adhesive side facing up until the water has evaporated from the surface.
- The silicone scar patch is adhesive again, once it is completely dry.
- The protective film can now be applied again.
- Finally, apply and use the silicone scar patch while it is dry.
- When the silicone scar patch is not being worn, it should be stored in a clean, dust-free space after washing and drying, with a protective foil on the adhesive side.

In the event of non-compliance with the hygiene instructions, skin irritation may occur due to germ growth underneath the silicone scar patch; the manufacturer may not be made liable for this.

Storage instructions and shelf life

A maximum shelf life of 6 years applies. The useful life of the CuraScar silicones is 4-6 weeks. Please store in a dry place at room temperature (max. 25 °C).

Special instructions

Page 70

Disposal instructions

Please enquire about regional disposal regulations.

Reporting obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies.

Service information

Do you have any questions?

If so, we would like to help – please call us or write to us. You can find our address on page 71.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2024 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd