amoena

BREAST FORMS & SHAPERS

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. For more than 45 years, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.

BREAST FORMS**Intended Purpose/Indication**

Target group: Women after various breast surgeries

Breast forms are required after partial or complete removal of the breast after breast cancer or other diseases and in the case of breast malformation or degeneration.

As well as providing a balanced look, the breast form is designed to maintain the symmetry of the body (preventing shoulder misalignment) by ensuring the breast form is the appropriate weight, and to simulate the movement of the natural breast.

Breast forms and breast shapers should only be worn once the scar area has fully healed.

Contraindications/Risks

No contraindications or risks are known or expected when using the above-mentioned products. In rare cases allergic reactions may occur. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin.

Material Composition

- Polyurethane film
- Polydimethylsiloxane and silicone oil
- Also, depending on the item: lightweight microspheres, temperature-regulating material (PCM Phase Change Material), additives and colour pigments

Instructions for Use

- We recommend wearing the breast forms in the pocket of a well-fitting pocketed bra. Breast forms can also be worn directly against the skin in a well-fitting non-pocketed bra.
- The breast forms and breast shapers tolerate chlorine and salt water without any problems. However, they can pick up odours over time (e.g. your body odour or odours of external origin such as chlorinated water), which can last for an indefinite period of time. We therefore recommend the Amoena Aqua Wave for water sports activities.
- We do not recommend wearing breast forms or shapers in solariums (sunbeds and tanning beds), saunas, steam baths or when scuba diving.
- It is advisable to check the fit of the breast form annually, since body measurements can change due to medication or life circumstances.

Care Instructions

- For the care of your Amoena breast form, we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed in clean water and dried with a soft towel. Body lotion, powder and natural body oils, can cause a film to form on the inside of the breast form.
- Please handle your breast form with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can damage the breast form. Breast forms made of lightweight silicone should be handled with particular care in order to avoid damaging the lightweight silicone. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Any type of discolouration or staining cannot be removed. However, it will not have any affect on the quality or function of the breast form.
- Some substances can damage the film coating of the breast form:
 - Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
 - Ethyl acetate and other esters
 - Acetone

Wait until the substances have been absorbed and wash your hands before touching your breast form. Disinfectants are permitted.

PRIFORM**Purpose**

Standard Priform and Premium Priform are lightweight primary care forms with fibre fill, designed to offer a balanced silhouette during the period directly after surgery or during radiotherapy.

This lightweight primary care form is not a long-term substitute for a silicone breast form.

Target group: Women after mastectomy

Indication

Primary care forms are designed to create a natural look for the patient after breast surgery during the healing period (6-8 weeks).

Contraindications/Risks

No contraindications or risks are known or expected. In rare cases allergic reactions may occur. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin.

Skin Care Instructions

- The care instructions are found on the care label of the Priform.
- Remove the fibre fill prior to washing.
- It is recommended that the fabric form and fibre fill are washed separately by hand and left to air dry.
- The fibre fill should only be replaced in the fabric form when the form and the fibre fill are both completely dry.

Instructions for Use Priform**Determining the Right Size**

The correct size of the Priform is based on the patient's bra size.

- Priforms can be worn in the pocket of a primary care textile.
- The volume can be altered by adjusting the polyester filling through the opening at the back.

Material Composition Priform

Standard Priform	Front:	Polyester
Premium Priform	Back:	Cotton
	Filling material:	Polyester

For percentage of material composition see care label.

Special Notes Priform

The Article 216 Premium Priform contains additional fibre fill for volume adjustment in the packaging.

SPECIAL INSTRUCTIONS

Protect from sunlight

Store in a dry place

Single patient – multiple uses

Breast form

Disposal Instructions

Dispose of the products in household waste.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies.

Service Information

Do you have any questions?

If so, we would like to help – please call us or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2022 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd
Amoena USA Corporation