amoena

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.

Purpose

Silicone pressure pads are used to support the treatment of lymphoedema and for scar therapy in combination with compression garments. When inserted into the garment, pressure pads help ensure an even distribution of pressure, maintain pressure stability, as well as a targeted increase in pressure. Additionally, the textured surface of the pressure pad promotes local mobilisation of fibrotic or oedematous tissue. Silicone can also positively influence scar maturation and enhance both the appearance and function of the scarred area.

Indications

Damage to venous or lymphatic vascular function or structure—whether post-operative, post-traumatic, or due to immobilisation-related congestion—as well as the prevention and treatment of problematic scars, can be addressed in combination with compression garments.

Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV, decompensated heart failure, septic phlebitis, phlegmasia cerulea dolens, acute erysipelas, acute bacterial, viral or allergic inflammations, untreated or advanced skin diseases/contact eczematia, untreated open wounds, swellings or extremities of an unknown origin, untreated malignant tumours.

Relative contraindications

Pronounced weeping dermatoses, malignant lymphoedema, peripheral arterial occlusive disease (PAOD) stage I/II, peripheral neuropathy in diabetes mellitus, sensory disorders of the extremities, compensated heart failure, skin infections, skin diseases (e.g. blistering dermatoses), intolerance to pressure or ingredients of the product, primary chronic polyarthritis. If there are open wounds in the treatment area, the wound area must first be covered with an appropriate dressing or bandage before the compression garment is applied. Amoena cannot accept any liability if these contraindications are not observed.

Side effects

No side effects are known if this product is used properly. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects. However, if negative changes such as skin irritation occur during prescribed use, please contact your doctor or specialist medical retailer immediately. If an intolerance to one or more components of this product is known, please consult your doctor before use. The manufacturer is not liable for any damage/injury caused due to improper handling or misuse.

Material composition

Pressure pad: Polydimethylsiloxane, polyurethane film, polypropylene protective film (PP)
Breast shell: Polydimethylsiloxane, Talcum

General instructions for use

Your specialist fitter will be happy to show you how to apply the product correctly. Only wear your pressure pad or breast shell as prescribed by your doctor. Rough, unfiled and cracked fingernails and/or sharp-edged rings etc. may damage the fine material. The pressure pad and breast shell should only be dispensed by trained specialists at medical retailers. In the event of damage, please contact your specialist retailer in confidence. Please do not repair this product yourself, as this may impair its quality and medical efficacy. For hygienic reasons, it is not suitable for the care of more than one patient. The pressure pad and breast shell should not be dried on a radiator, in a microwave, or in direct sunlight. Continuous use of disinfectants should be avoided. Slight discoloration may occur over time but does not affect the function or effectiveness of the pressure pad or breast shell.

General Care Instructions

Please clean the pressure pad/breast shell before each use, **especially before the first use.**

1. Apply three drops of Amoena Soft Cleanser to the dry, textured surface and then moisten the soft-brush with lukewarm water. (Fig. 1).
2. Now clean the structured surface thoroughly under pressure using only the soft brush. (Fig. 2).
3. Then rinse with plenty of lukewarm water (Fig. 3).
4. Finally, allow to air dry or blow dry on a medium setting.

LYMPH FLOW PRESSURE PAD

Application instructions for pressure pads

The pressure pad can be inserted under the garment textile in the required place. The textured surface is directed towards the skin. The pressure pad can be cut to according to your needs. It is supplied with a sample size of Amoena Soft Cleanser 30ml and an Amoena Soft Brush. Please clean the pressure pad before each application, **especially before the first application.**

LYMPH FLOW BREAST SHELL

Application instructions for breast shell

It is supplied with a sample size of Amoena Soft Cleanser 30ml and an Amoena Soft Brush.
Please clean the breast shell before each use, **especially before the first application.**

1. The breast shell can be cut with scissors according to the breast size. Use the lines provided in the breast shell. To determine the size, place the breast shell on your chest and select the line that corresponds to your breast size. (Fig 4).
2. Place the breast shell directly on the affected breast tissue, with the textured surface facing the skin (Fig 5). The small arrow in the centre of the inside of the shell should point upwards.
3. For optimal effect, the breast shell should be worn under a bra.

Storage instructions and shelf life

Pressure pad and breast shell: Store in a dry place and protect from sunlight. The maximum shelf life is 70 months and the periode of use of the pressure pad is 6 months. This depends on the correct handling and use of the products to the usual extent. The total life time of the pressure pad is therefore limited to 78 months. The shelf life is printed with the hourglass symbol on the box label.

Special notes

Page 70.

Disposal instructions

Products should be disposed of in household waste.

Reporting requirement

In the event of any product-related complaints, please contact your specialist medical retailer directly. Only serious incidents that may lead to a significant deterioration of health or death must be reported to the manufacturer and the competent authority of the member state. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation shall apply.

Service information

Still have any questions?

We will be happy to advise you – please give us a call or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2025 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd