amoena

BREAST FORMS & SHAPERS

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.

Intended Purpose/Indication

Target group: Women after various breast surgeries

Breast forms are required after partial or complete removal of the breast after breast cancer or other diseases and in the case of breast malformation or degeneration. As well as providing a balanced look, the breast form is designed to maintain the symmetry of the body (preventing shoulder misalignment) by ensuring the breast form is the appropriate weight, and to simulate the movement of the natural breast. Breast forms and breast shapers should only be worn once the scar area has fully healed.

Contraindications/Risks/Side effects

No contraindications or risks are known or expected when using the above-mentioned products. In rare cases allergic reactions may occur. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

Material Composition

- Polyurethane film
- Polydimethylsiloxane and silicone oil
- Also, depending on the item: lightweight microspheres, temperature-regulating material (PCM Phase Change Material), additives and colour pigments

General Instructions for Use

- We recommend wearing the breast forms in the pocket of a well-fitting specialist bra. Contact breast forms and breast shapers adhere directly to the skin.
- The breast forms and breast shapers tolerate chlorine and salt water without any problems. However, they can pick up odours over time (e.g. your body odour or odours of external origin such as chlorinated water), which can last for an indefinite period of time. Due to the valve, we do not recommend you wear Adapt Air when swimming. We therefore recommend the Amoena Aqua Wave and Balance Aqua Wave for water sports activities.
- We do not recommend wearing breast forms or shapers in solariums (sunbeds and tanning beds), saunas, steam baths, in spas, saunas or hot tubs or when scuba diving.
- It is advisable to check the fit of the breast form annually, since body measurements can change due to medication or life circumstances.
- Avoid direct contact with perfume, hot surfaces and hot water

General Care Instructions

- For the care of your Amoena breast form (does not apply to the Amoena Contact prosthesis, see page 16), we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed in clean water and dried with a soft towel. Body lotion, powder and natural body oils, can cause a film to form on the inside of the breast form.
- Please handle your breast form with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can damage the breast form. Breast forms made of lightweight silicone should be handled with particular care in order to avoid damaging the lightweight silicone. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Any type of discolouration or staining cannot be removed. However, it will not have any affect on the quality or function of the breast form.
- Some substances can damage the film coating of the breast form:
 - Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
 - Ethyl acetate and other esters
 - Acetone
 Wait until the substances have been absorbed and wash your hands before touching your breast form. Disinfectants are permitted.
- Product-specific care and application instructions for Amoena Contact and Adapt Air prostheses can be found on pages 14-17

CONTACT BREAST FORMS

All Contact products are delivered with a sample size Amoena Soft Cleanser a soft brush, a protective film and a cover pad.

Instructions for Use Contact Breast Forms

- The adhesive strength of the breast form can vary depending on your skin type (oily or dry). It is also dependent on how much you sweat, your level of physical exertion and the condition of the scar area.
- However, selective adhesion is sufficient to reduce strain on your shoulder and neck.
- The Contact breast form can be worn when playing sport, but adhesion may be reduced due to the build-up of sweat. The adhesive breast form is chlorine-resistant and salt-resistant. However, it should be cleaned directly after use in clean water with the Amoena Soft Cleanser.
- Wear the Amoena Contact every day from the beginning. The more regularly the breast form is worn, the better your skin will accept the adhesive.
- In the case of extensive and intensive wear, sweat and small skin particles will build up on the adhesive surface in the course of the day. This means adhesion will slowly decrease.
- Thorough cleaning of the adhesive surface reactivates the adhesion. Putting the breast form back on after cleaning gives you a feeling of security again.
- If your skin tends towards dryness, apply moisturising skin care after wearing the Contact breast form.
- For each Contact breast form and Contact breast shaper, a 3-month exchange right applies. If you struggle to achieve adhesion within the first 3 months, you can exchange it for a bra-supported Amoena product.

- The protective film protects the adhesive surface during storage. Please remove it before wearing. The cover pad can be used to cover the adhesive surface so that the prosthesis is not self-adhesive but can be worn in the bra pocket.

Putting On the Contact Breast Form

- First clean your skin carefully. Then let the skin dry fully. The drier and cleaner your skin is, the better the adhesion of the breast form. Please do not touch the adhesive surface of the breast form, as this can impair the adhesion.
- Now put on your bra and check the fit. Next pull your bra strap completely off your shoulder and arm and lean forwards.
- Using the hand from your non-operated side, take hold of the bra strap and pull the cup away from the body.
- Take the breast form in the other hand and place it in the cup of the bra. The upper corner of the breast form should be close to the bra strap. Stay leaning forwards. Make sure that the breast form sits well in the bra. See diagram 1 page 3
- Cup the outer edge of your bra with your free hand and slowly straighten up. As you do so, carefully press the breast form onto your body.
- Now pull the bra strap over your shoulder and press the breast form firmly into place again. See diagram 2 page 3
- Check the fit. If the breast form is not well positioned, lean forwards and start over.

Taking Off the Contact Breast Form

- Cup the breast form with one hand as completely as possible, so that your fingertips are near the edge, and lean forwards until you feel the weight of the breast form in your hand. Starting at a point some distance from the scar, slowly pull the breast form off the body.
- If the adhesive surfaces accidentally stick together as they are removed, they can be easily separated again under running water.
- Please be careful with long fingernails. These can damage the coating of the breast form.

General Care Instructions Contact Breast Forms

- Only use Amoena Soft Cleanser to clean your self-adhesive Contact breast form. It reactivates the adhesive and therefore enables the best adhesion..
- Skin texture and skin evaporations caused by e.g. medications and care products can affect the adhesive strength. Please clean your breast form daily and thoroughly to achieve optimal and long-term adhesion.
- Small clumps that form on the adhesive surface should not be removed. Do not use any other cleaning agents such as abrasive powder or implements such as scissors or tweezers. Clean the adhesive surface with the Amoena Soft Cleanser and the special brush.
- Only the special brush supplied should be used for cleaning and should be replaced after 12 months at the latest

IMPORTANT: Cleaning the Adhesive Surface

1. Spread three drops of the Amoena Soft Cleanser to the dry adhesive surface and then moisten the special brush with lukewarm water. See diagram 3 Contact page 3
2. Using only the special brush, clean the adhesive surface completely for approx. 2 minutes, applying pressure and using circular movements until the cleanser foams. See diagram 4 Contact page 3
3. Then rinse with plenty of lukewarm water.
4. Leave to air dry or dry with a hair dryer on a moderate setting. Do not dry the adhesive surface with a towel.
5. Place the clean and dry protective film on the dried and cleaned adhesive surface and smooth out the edges.

ADAPT BREAST FORMS

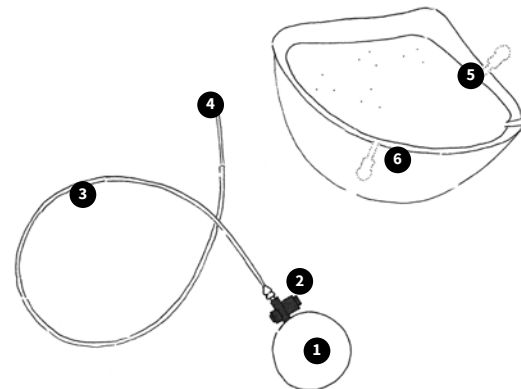
All Adapt Air products are delivered with a pump. All Adapt Air Contact products are delivered with a pump, a sample size Amoena Soft Cleanser a soft brush, a protective film and a cover pad.

General Instructions for Use Adapt Breast Forms

Amoena Adapt Air are breast forms with an integrated air chamber which can be individually adjusted to fit the chest wall and the body's natural silhouette. A special pump is required to fill the chamber with air or drain the air from the chamber. Small quantities of silicone oil, which can form on the cannula of the pump, do not impair the function and quality of the breast form or pump. Simply wipe away the oil with a clean cloth.

Adjusting Adapt Air

1. Lay out the breast form and pump ready for use. Breast form and pump are separate. Please leave the protective film on the adhesive surface of the Adapt Air Contact. See diagram 5 page 3.
2. Place the pump (1) under the bra strap on the opposite side so that you have both hands free. Pull the bra strap off the shoulder of the operated side. Diagram 6 page 3.
3. Connect the pump by removing the grey tag (if present) and inserting the cannula (4) into the opening (5), for Adapt Air Contact (6) of the breast form marked with a white arrow. Open the valve inlet by gently squeezing your thumb and index finger together. Guide the cannula completely into the valve. Now remove the protective film from the Adapt Air Contact. See diagram 7 page 4.
4. Bend forwards slightly and insert the breast prosthesis into the bra cup. Make sure that the upper corner of the prosthesis is close to the base of the strap and that the tube is correctly connected to the prosthesis. See diagram 8 page 4
5. While you bring the bra strap back into the correct position, straighten up and gently press the prosthesis against the chest wall with your hand. See diagram 9 page 4
6. Now, adjust the breast form to your needs by pressing the pump ball (1). To drain the air, press the black button on the drain valve (2) and lightly press the breast form against the chest wall. See diagram 10 page 4
7. Please note: In order to insert or drain air, the cannula must be inserted completely into the valve.
8. As soon as the correct volume is set, remove the cannula carefully from the valve. The air remains in the air chamber. See diagram 11 page 4

1. Pump ball
 2. Drain valve
 3. Hose
 4. Cannula
 5. Valve opening
 6. Valve opening
- Adapt Air Contact**

9. Now the breast form has been adapted to your individual needs. Before wearing the non-adhesive Adapt Air prosthesis, wipe the valve with a clean cloth as there may still be a residue of silicone oil from the cannula. Then place the prosthesis in the pocket of the bra. See diagram 12 page 4.
10. With the adhesive Adapt Air Contact, clean the valve area with a tissue and check the fit of the prosthesis. If the prosthesis does not fit well, remove the prosthesis (see page 15) and start fitting from the beginning.

General Care Instructions for Adapt Air Breast Forms

- When cleaning, ensure that no water enters the valve.
- Due to the valve, we do not recommend you wear Adapt Air when swimming.
- Changes in air pressure can lead to minimal, invisible changes in volume. Adjust the volume with the pump if necessary.
- Use only the pump supplied with the breast form for any adjustments.
- The special care and cleaning instructions for the adhesive surface of the Adapt Air Contact can be found on page 15.

Pump Materials

Hollow body: Soft PVC, phthalate-free
 Silicone hose: Silicone
 PP tip: PP
 Pressure drain valve: ABS

General Care Instructions for Pump

Clean the pump ball, the hose if necessary, and the drain valve with a mild soap solution. Disinfectants which contain alcohol can damage the pump. The proper function of the pump is guaranteed between -10°C and +40°C, while the maximum permissible temperature lies between -30°C and +100°C. It can be disinfected by boiling. In this case, separate the component parts from the pump ball and boil it in ample water for a short time.

Hazard Warning

Please keep away from children. The product may contain small parts such as adapters or valves. If these parts are swallowed, there is a risk of suffocation.

Storage instructions and shelf life

We recommend keeping your prosthesis horizontal when not being worn, storing it in the box supplied. Amoena breast forms have a shelf life of 6 years (expiry date on label), but they can continue to be used for up to 2 years.

Special Instructions

Page 158

Disposal Instructions

Dispose of the products in household waste.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies.

Service Information

Do you have any questions? If so, we would like to help – please call us or write to us. You can find our address on page 157.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2024 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd