amoena

## **CUSTOM FORM**

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Gebrauchsanweisung.  
Instructions for use.  
Mode d'emploi.  
Instrucciones de uso.  
使用説明.

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**Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. For more than 45 years, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.**

**Intended Purpose/Indication**

Target group: Women after various breast surgeries.

Breast forms are required after partial or complete removal of the breast after breast cancer or other diseases and in the case of breast malformation or degeneration. As well as providing a balanced look, the breast form is designed to maintain the symmetry of the body (preventing shoulder misalignment) by ensuring the breast form is the appropriate weight, and to simulate the movement of the natural breast. Breast forms and shapers should only be worn after the scars have healed completely. Custom breast forms are required for the definitive restoration due to anatomically or structurally necessary individual adaptation and with the need to relieve the skin or scar area against pressure and tensile forces. The Amoena Custom is individually designed and manufactured according to the individual needs of each patient. Breast forms and breast shapers should only be worn once the scar area has fully healed. Medical indications for the custom breast form are:

- Structural damage with loss of substance of the mammary gland after surgery or radiation in case of tumor diseases (e.g. after breast removal, partial removal or radiation fibrosis)
- Structural damage with loss of substance of the mammary gland after surgery for other diseases such as phlegmon, abscesses (e.g. after partial or complete removal of the breast)
- Congenital structural damage with asymmetry of the breasts such as the Poland syndrome

Asymmetric breast prostheses are indicated if anatomical or structural conditions require asymmetrical care.

**Contraindications/risks/side effects**

No contraindications or risks are known or expected when using the above-mentioned products. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

**Material composition**

Polyurethane film, polydimethylsiloxane, silicone oil

Also, depending on the product feature: lightweight microspheres, temperature regulating material (PCM Phase Change Material), additives and color pigments

**General instructions for use**

- We recommend wearing the breast forms in the pocket of a well-fitting specialist bra. Shapers can also be worn directly against the skin in a well-fitting non-pocketed bra. Custom +++ breast forms and breast shapers with self-adhesive back adhere directly to the skin.

- The breast forms and breast shapers tolerate chlorine and salt water without any problems. However, they can pick up odours over time (e.g. your body odour or odours of external origin such as chlorinated water), which can last for an indefinite period of time. Therefore, we recommend the Amoena Aqua Wave for water sports activities.
- We do not recommend wearing breast forms or shapers in solariums (sunbeds and tanning beds), saunas, steam baths, in spas, saunas or hot tubs or when scuba diving.
- It is advisable to check the fit of the breast form annually, since body measurements can change due to medication or life circumstances.
- Avoid direct contact with perfume, hot surfaces and hot water

### General care instructions

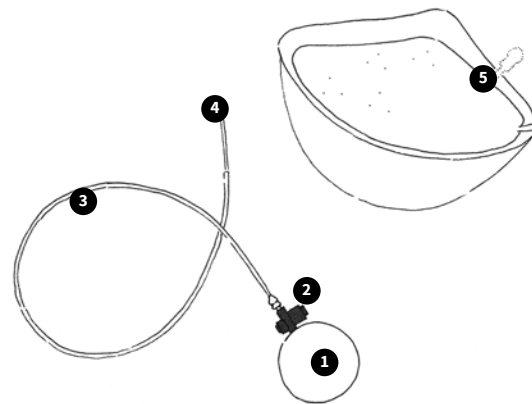
- For the care of your Custom (does not apply to the Amoena Contact prosthesis, see page 16), we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed with clean water and dried with a soft towel. Body lotion, powder and natural body oils, can cause a film to form on the inside of the breast form.
- Please handle your Custom with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can damage the breast form. Breast forms made of lightweight silicone should be handled with particular care in order to avoid damaging the lightweight silicone. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Any type of discolouration or staining cannot be removed. However, it will not have any effect on the quality or function of the breast form.
- Some substances can damage the film coating of your Custom:
  - » Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
  - » Ethyl acetate and other esters
  - » Acetone

Wait until the substances have been absorbed and wash your hands before touching your breast form. Disinfectants are permitted.

Product-specific care and application instructions for Amoena Custom Contact +++ and Custom +++ prosthesis can be found on pages 14-18.

### CUSTOM BREAST FORMS WITH ADAPT AIR TECHNOLOGY (303 CUSTOM +++)

Custom with Adapt Air Technology contains an integrated air chamber, which can be individually adjusted to fit the chest wall and the body's natural silhouette. A special pump is required to fill the chamber with air or drain the air from the chamber. Small quantities of silicone oil, which can form on the cannula of the pump, do not impair the function and quality of the breast form or pump. Simply wipe away the oil with a clean cloth.

1. Pump ball
2. Drain valve
3. Hose
4. Cannula
5. Valve opening

1. Lay out the breast form and pump ready for use. Breast form and pump are separate. Diagram 5 page 03.
2. Place the pump (1) under the bra strap on the opposite side so that you have both hands free. Pull the bra strap off the shoulder of the operated side. Diagram 6 page 03.
3. Connect the pump by removing the grey tag (if present) and inserting the cannula (4) into the opening (5) of the breast form marked with a white arrow. Open the valve inlet by gently squeezing your thumb and index finger together. Guide the cannula completely into the valve. Diagram 7 page 04.
4. Put the breast form into the bra cup. Ensure that the hose is connected correctly with the breast form. Diagram 8 page 04.
5. Bring the bra strap back in the correct position. Diagram 9 page 04.
6. Now, adjust the breast form to your needs by pressing the pump ball (1). To drain the air, press the black button on the drain valve (2) and lightly press the breast form against the chest wall. Diagram 10 page 04.
7. Please note: In order to insert or drain air, the cannula must be inserted completely into the valve.
8. As soon as the correct volume is set, remove the cannula carefully from the valve. The air remains in the air chamber. Diagram 11 page 04.

- Now the breast form has been adapted to your individual needs. Remove it from the cup and clean the valve area with a tissue, as small amounts of care oil from the cannula (tube) may still remain. Then place the breast form in the pocket of the bra. Sketch 12 on page 04.

#### Care instructions

- When cleaning, ensure that no water enters the valve.
- Due to the valve, we do not recommend you to wear your Custom when swimming.
- Changes in air pressure can lead to minimal, invisible changes in volume. Adjust the volume with the pump if necessary.
- Use only the pump supplied with your Custom for any adjustments.

#### Care instructions for pump

Clean the pump ball, the hose if necessary, and the drain valve with a mild soap solution. Disinfectants that contain alcohol can damage the pump. The proper function of the pump is guaranteed between -10°C and + 40°C, while the maximum permissible temperature lies between -30°C and +100°C. It can be disinfected by boiling. In this case, separate the component parts from the pump ball and boil it in ample water for a short time.

#### Pump materials

Pump Ball: Soft PVC, phthalate-free  
 Drain valve: ABS  
 Hose: Silicone  
 Cannula: PP

#### CUSTOM BREAST FORMS WITH A SELF-ADHESIVE BACK (304 CUSTOM CONTACT +++)

All Contact products are delivered with a sample size Amoena Soft Cleanser a soft brush, a protective film and a cover pad.

#### Instructions for Use Custom Contact Breast Forms

- The adhesive strength of the breast form can vary depending on your skin type (oily or dry). It is also dependent on how much you sweat, your level of physical exertion and the condition of the scar area.
- However, selective adhesion is sufficient to reduce strain on your shoulder and neck.
- The Custom Contact breast form can be worn when playing sport, but adhesion may be reduced due to the build-up of sweat. The adhesive breast form is chlorine-resistant and salt-resistant. However, it should be cleaned directly after use in clean water with the Amoena Soft Cleanser.
- Wear the Amoena Custom Contact every day from the beginning. The more regularly the breast form is worn, the better your skin will accept the adhesive.

- In the case of extensive and intensive wear, sweat and small skin particles will build up on the adhesive surface in the course of the day. This means adhesion will slowly decrease.
- Thorough cleaning of the adhesive surface reactivates the adhesion. Putting the breast form back on after cleaning gives you a feeling of security again.
- If your skin tends towards dryness, apply moisturising skin care after wearing the Contact breast form.
- The protective film protects the adhesive surface during storage. Please remove it before wearing. The cover pad can be used to cover the adhesive surface so that the prosthesis is not self-adhesive but can be worn in the bra pocket.

#### Putting On the Custom Contact Breast Form

- First clean your skin carefully. Then let the skin dry fully. The drier and cleaner your skin is, the better the adhesion of the breast form. Please do not touch the adhesive surface of the breast form, as this can impair the adhesion.
- Now put on your bra and check the fit. Next pull your bra strap completely off your shoulder and arm and lean forwards.
- Using the hand from your non-operated side, take hold of the bra strap and pull the cup away from the body.
- Take the breast form in the other hand and place it in the cup of the bra. The upper corner of the breast form should be close to the bra strap. Stay leaning forwards. Make sure that the breast form sits well in the bra. See diagram 9 page 04
- Cup the outer edge of your bra with your free hand and slowly straighten up. As you do so, carefully press the breast form onto your body.
- Now pull the bra strap over your shoulder and press the breast form firmly into place again. See diagram 10 page 04
- Check the fit. If the breast form is not well positioned, lean forwards and start over.

#### Taking Off the Custom Contact Breast Form

- Cup the breast form with one hand as completely as possible, so that your fingertips are near the edge, and lean forwards until you feel the weight of the breast form in your hand. Starting at a point some distance from the scar, slowly pull the breast form off the body.
- If the adhesive surfaces accidentally stick together as they are removed, they can be easily separated again under running water.
- Please be careful with long fingernails. These can damage the coating of the breast form.

#### General Care and cleaning Instructions Custom Contact Breast Forms

- Only use Amoena Soft Cleanser to clean your self-adhesive Contact breast form. It reactivates the adhesive and therefore enables the best adhesion.
- Skin texture and skin evaporations caused by e.g. medications and care products can affect the

adhesive strength. Please clean your breast form daily and thoroughly to achieve optimal and long-term adhesion.

- Small clumps that form on the adhesive surface should not be removed. Do not use any other cleaning agents such as abrasive powder or implements such as scissors or tweezers. Clean the adhesive surface with the Amoena Soft Cleanser and the special brush.
- Only the special brush supplied should be used for cleaning and should be replaced after 12 months at the latest

#### **IMPORTANT: Cleaning the Adhesive Surface**

1. Spread three drops of the Amoena Soft Cleanser to the dry adhesive surface and then moisten the special brush with lukewarm water. See diagram 11 page 04
2. Using only the special brush, clean the adhesive surface completely for approx. 2 minutes, applying pressure and using circular movements until the cleanser foams. See diagram 12 page 04
3. Then rinse with plenty of lukewarm water.
4. Leave to air dry or dry with a hair dryer on a moderate setting. Do not dry the adhesive surface with a towel.
5. Place the clean and dry protective film on the dried and cleaned adhesive surface and smooth out the edges.

#### **Hazard warning pump**

Please keep away from children. The product may contain small parts such as adapters or valves. If these parts are swallowed, there is a risk of suffocation.

#### **Special instructions**

Page 38

#### **Disposal instructions**

Dispose of the products in household waste.

#### **Reporting obligation**

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

#### **Warranty**

The statutory warranty obligation applies.

#### **Service information**

Do you have any questions? If so, we would like to help – please call us or write to us. You can find our address on page 39.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2023 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,