amoena

RECOVERY CARE
CURASUPPORT COMPRESSION
PATTY SB FC

Instructions for use.
Mode d'emploi.
Instrucciones de uso.

English	ENG	4
Français	FR	6
Español	ES	10

Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.

PATTY SB FC

Art. No.		Article
44938		Patty SB FC (Black)

Intended Purpose /Indication

Target group: Women after various breast surgeries

Compression textiles are designed to keep the operated breast or breasts immobile and/or stabilised during the healing phase.

Compression textiles

- can support wound healing after breast surgery.
- can stabilise the shape and function of the operated breast.
- can protect sensitive scar tissue and reduce irritation.
- can minimise the risk of postoperative oedema.
- can reduce postoperative swelling/pain.

Contraindications/Risks

Relative contraindications: In the case of the following clinical symptoms, it is up to your doctor to decide whether a compression textile can be used:

- Skin diseases
- Very sensitive scar area in the surgical zone
- Radiodermatitis
- Known material allergy (contact dermatitis)

If there are open wounds in the treatment area, the wound area must first be covered with an appropriate dressing or bandage before the compression garment is applied. Amoena cannot accept any liability if these contraindications are not observed.

Side effects

If the wrong size is selected, this may lead to localised pressure hypersensitivity and disorders of blood flow and/or lymphatic flow. A contraindication exists if there is a known allergy to one of the

components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

General Care Instructions

- The care instructions can be found on the care label of the compression textile.
- Before washing, close the zip and hook-and-loop fastener as well as hooks and eyes.
- It is recommended that the textile is washed in a laundry bag.
- If the compression textile is washed with other textiles, it is recommended that similar colours should be washed together.

Material Composition

Polyamide, elastane

For percentages of material composition, please read the care label.
Free of latex, silicone and nickel.

Instructions for Use

To ensure an optimal healing process, sufficient immobilisation and stabilisation of the treated breast tissue is highly advisable. Optimal fixation/stabilization of the operated breast tissue is also guaranteed with double sizes and universal cups. Depending on the individual patient's medical indication, the Seamless Compression Bra can be combined with any compression belt. Please do not place on open wounds. Seamless wide bottom band fits gently around catheters and and securely holds the Amoena Drain Pouch (sold separately) if needed.

Determining the Correct Bra Size

The patient's underbust circumference when standing, is determined by using a standard proprietary metric measuring tape. Using the table below, the underbust measurement obtained, can be allocated to an underbust size. The unique seamless knitting technology, can provide universal cup sizes and individually adjusts to the breast size and shape, from A – F cup.

Underbust measurement	58–67	68–77	78–87	88–97	98–107	108–117	118–127
Size	60/65	70/75	80/85	90/95	100/105	110/115	120/125
	XS	S	M	L	XL	2XL	3XL

Storage advice and durability

A shelf life of 36 months and a period of use of 6 months apply. The total life time is therefore limited to 42 months The period of use of compression textiles is 6 months and depends on the correct handling such as following care instructions, donning and doffing (or “dressing and undressing the compression product”) and the usage of the products to a normal extent. The shelf life of the medical device is shown by the hourglass symbol on the packaging. When properly treated, the compression textiles retain their medicinal effect over the full period of 6 months. If your body dimensions should change - due to your medical analysis and individual life circumstances – a new measurement and a new fitting is advisable even before the 6-month period has expired

Special Instructions

Page 15

Disposal Instructions

Dispose of the products in household waste.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies.

Service Information

Do you have any questions?
If so, we would like to help – please call us or write to us.
You can find our address on page 14.

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This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Canada
Amoena Canada Inc. 101 – 6790
Century Avenue Mississauga
Ontario L5N 2V8
www.amoena.ca

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2025 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,