Soft Cleanser do mycia protezy

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Soft Cleanser

Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.

Indications: For cleaning the adhesive surface of your Amoena Contact breast form and for generally cleaning all Amoena breast forms.

Through daily cleaning activates the adhesion of the adhesive surface and is crucial for the Contact breast form to sit securely and to preserve the adhesive surface.

Cleaning:

1. Apply three drops Amoena Soft Cleanser undiluted onto the dry Amoena Contact adhesive surface.
2. Moisten the Amoena Special Brush with water. Clean the adhesive surface vigorously for approximately 3 mins. using circular motions.
3. Important! Rinse it off thoroughly with plenty of lukewarm water.
4. Leave to air dry or dry with a hair dryer on a moderate setting.

Storage Instructions:

Store the product dry at room temperature and keep out of direct sunlight. After opening the bottle, you should use it up within 12 months.

There are no known contraindications or risks. In rare cases, residual risks such as allergic reactions may occur. If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Special Instructions:

Do not wear the Contact breast form until the scar is completely healed, till the skin is no longer irritated and all therapies such as chemotherapy, radiotherapy, etc. have been completed. The adhesive surface should not come into contact with broken or damaged skin.

Disposal Instructions:

Dispose of the completely empty bottle with your normal household waste.

Do you have any questions?

If so, we would like to help – please call us or write to us:
Amoena UK Ltd
1 Eagle Close, Chandlers Ford, Eastleigh,
Hampshire, SO53 4NF
Tel. +44 (0) 2380 270345
www.amoena.co.uk

Amoena USA Corporation
1000 Cobb Place Blvd.
Ste. 200, Kennesaw, GA 30144,
United States
Phone: +1 (800) 741-0078
www.amoena.us

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH. © 2023 All rights reserved
Amoena Sweden AB