# amoena

## **BREAST FORMS & SHAPERS**

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**Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.**

## BREAST FORMS

### Intended Purpose/Indication

Target group: Women after various breast surgeries

Breast forms are required after partial or complete removal of the breast after breast cancer or other diseases and in the case of breast malformation or degeneration.

As well as providing a balanced look, the breast form is designed to maintain the symmetry of the body (preventing shoulder misalignment) by ensuring the breast form is the appropriate weight, and to simulate the movement of the natural breast.

Breast forms and breast shapers should only be worn once the scar area has fully healed.

### Contraindications/Risks/Side effects

No contraindications or risks are known or expected when using the above-mentioned products. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

### Material Composition

- Polyurethane film
- Polydimethylsiloxane and silicone oil
- Also, depending on the item: lightweight microspheres, temperature-regulating material (PCM Phase Change Material), additives and colour pigments

### General Instructions for Use

We recommend wearing the breast forms in the pocket of a well-fitting pocketed bra. Breast forms can also be worn directly against the skin in a well-fitting non-pocketed bra.

- The breast forms and breast shapers tolerate chlorine and salt water without any problems. However, they can pick up odours over time (e.g. your body odour or odours of external origin such as chlorinated water), which can last for an indefinite period of time. We therefore recommend the Amoena Aqua Wave for water sports activities.
- We do not recommend wearing breast forms or shapers in solariums (sunbeds and tanning beds), saunas, steam baths, in spas, saunas or hot tubs or when scuba diving.

- It is advisable to check the fit of the breast form annually, since body measurements can change due to medication or life circumstances.
- Avoid direct contact with perfume, hot surfaces and hot water.

### General Care Instructions

- For the care of your Amoena breast form, we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed in clean water and dried with a soft towel. Body lotion, powder and natural body oils, can cause a film to form on the inside of the breast form.
- Please handle your breast form with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can damage the breast form. Breast forms made of lightweight silicone should be handled with particular care in order to avoid damaging the lightweight silicone. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Any type of discolouration or staining cannot be removed. However, it will not have any affect on the quality or function of the breast form.
- Some substances can damage the film coating of the breast form:
  - Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
  - Ethyl acetate and other esters
  - Acetone

Wait until the substances have been absorbed and wash your hands before touching your breast form. Disinfectants are permitted.

## LEISURE FORM

### Intended Purpose

Breast shaper made of lightweight foam for occasional use during leisure time or overnight. It ensures visual body symmetry after a mastectomy and keeps the remaining breast in position when lying down/sleeping.

### Material Composition

Weighted foam: polyurethane, barium sulphate (7727-43-7)

Cover: polyamide, elastane

**Instructions for Use Leisure Form**

- This foam form is not a substitute for wearing a silicone breast form, which is necessary for weight compensation and an upright posture.
- It is not suitable for sports activities.
- The form can be worn in a pocketed top, bra or nightdress to restore body symmetry.
- Detectors at the airport may react to the barium sulphate filling material.

**Care Instructions Leisure Form**

- The Amoena Leisure Form is washable.
- Remove the fabric cover from the form before washing.
- Only clean the foam form with a damp cloth. Do not launder in a washing machine or place in water.
- Wash the fabric cover either in the washing machine on a delicate wash cycle or by hand with a mild detergent at a maximum of 30°C.
- Lay the cover flat to dry – do not place in the tumble dryer.
- As soon as the fabric cover is completely dry, turn the cover inside out. Lay the front of the cover on the front side of the form and then carefully pull the back of the cover over the form.
- You can then put the breast shaper back in the pocket of your Amoena sleepwear or a well-fitting bra

**(BALANCE) AQUA WAVE****Intended Purpose**

Transparent silicone breast shaper for all activities in and on the water.

**Material Composition**

- Polyurethane film
- Polydimethylsiloxane and silicone oil

**Instructions for Use (Balance) Aqua Wave**

- The Amoena Aqua Wave can be worn in the integrated bra pocket of a bathing costume or bikini.
- This breast shaper is not a longterm substitute for wearing a silicone breast form, which is necessary for weight compensation and an upright posture.
- 

**PRIFORM****Purpose**

Standard Priform and Premium Priform are lightweight primary care forms with fibre fill, designed to offer a balanced silhouette during the period directly after surgery or during radiotherapy. This lightweight primary care form is not a long-term substitute for a silicone breast form. Target group: Women after mastectomy

**Indication**

Primary care forms are designed to create a natural look for the patient after breast surgery during the healing period (6-8 weeks).

**Contraindications/Risks**

No contraindications or risks are known or expected when using the above-mentioned products. A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

**Material Composition Priform**

Standard Priform/Premium Priform: Front: Polyester; Back: Cotton;  
Filling material: Polyester  
For percentage of material composition see care label.

**Instructions for Use Priform**

Determining the Right Size:

The correct size of the Priform is based on the patient's bra size.

- Priforms can be worn in the pocket of a primary care textile.
- The volume can be altered by adjusting the polyester filling through the opening at the back. The Article 216 Premium Priform contains additional fibre fill for volume adjustment in the packaging.

**Care Instructions Priform**

- The care instructions are found on the care label of the Priform.
- Remove the fibre fill prior to washing.
- It is recommended that the fabric form and fibre fill are washed separately by hand and left to air dry.
- The fibre fill should only be replaced in the fabric form when the form and the fibre fill are both completely dry.

### Storage instructions and shelf life

We recommend keeping your prosthesis horizontal when not being worn, storing it in the box supplied. Amoena breast forms have a shelf life of 6 years (expiry date on label), but they can continue to be used for up to 2 years.

### Special Instructions

Page 122

### Disposal Instructions

Dispose of the products in household waste.

### Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

### Warranty

The statutory warranty obligation applies.

### Service Information

Do you have any questions?

If so, we would be happy to advise you – please call us or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2024 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd